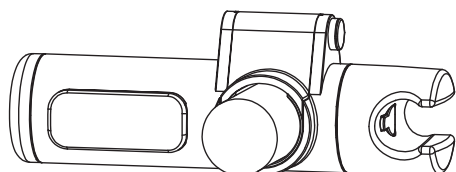
INSTRUCTION MANUAL MANUEL D'INSTRUCTION

GBH0010

**Shower arm
Support de douche**

AVIS

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WARNING

We recommend cleaning the product with soft soaps. DO NOT USE cleaning products made of: ketones, hydrocarbons, acids, bases, esters and ethers. Disinfection must be done according to the recommendations of the disinfectant. After use of a cleaning product, rinse with clear water.

Customer Service - Monday to Friday

question@dimex360.com

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question@dimex360.com

LIMITED WARRANTY

akuaplus® products have been designed to meet high-quality standards and come with a one year limited warranty against manufacturing or material defect.. This product is guaranteed if used in a normal way and the warranty only applies to the original buyer. Any warranty becomes void if the product was used in a non-recommended way or has been abused. Dimex360 Inc. is not responsible for labour costs, transportation, injuries or any loss or damage incurred during installation, repair or replacement of the product, including any consequential or indirect damage.

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Designed, imported and packaged exclusively by Dimex360 Inc.
Conçu, importé et emballé exclusivement par Dimex360 inc.
www.dimex360.com • question@dimex360.com

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
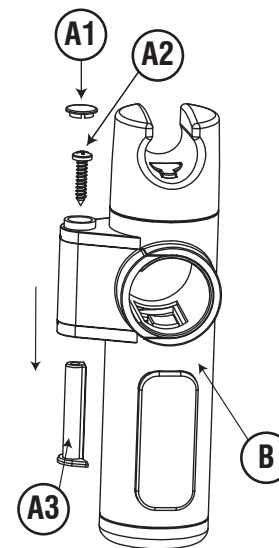
1. Pull out the cap (A1), the screw (A2) and the pin (A3) from the slider body (B).

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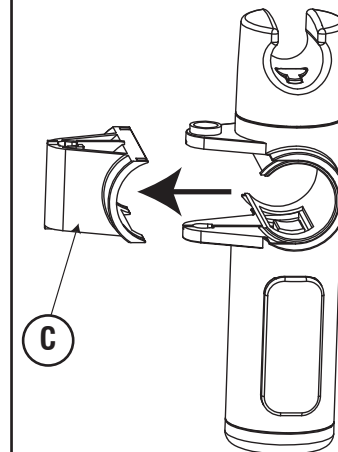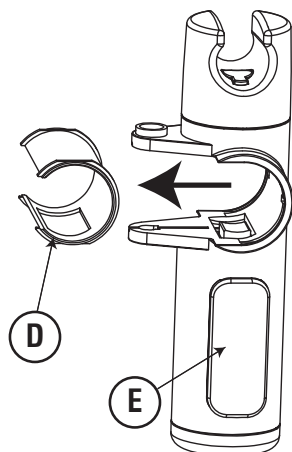
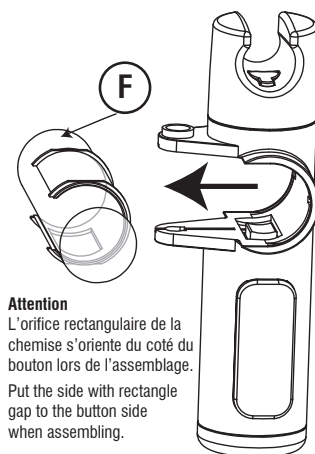
4. Put the liner on the grab bar (F), then push the button (E) and place the slider body over the liner.

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
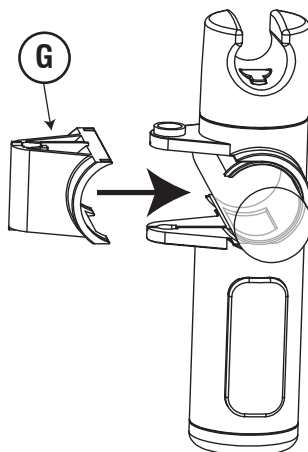
5. Put the back cover (G) in place on the slider body.


6. Insérez la tige (H).

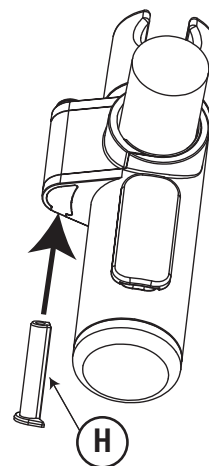6. Insert the pin (H).

7. Vissez la vis (J) et placez le capuchon (K).

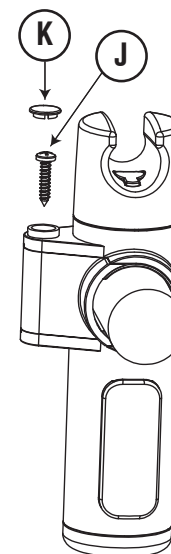7. Screw the screw (J) and put on the cover (K).

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8. Finished product.